

# Signaspray® Electrode Solution and Skin Prep

REF 18-04, 18-25, 18-28

- EN Instructions for use
- ES Instrucciones de uso
- DE Gebrauchsanweisung
- FR Mode d'emploi
- IT Istruzioni per l'uso
- PT Instruções de utilização
- DA Brugsanvisning
- FI Käyttöohjeet
- NL Gebruiksaanwijzing
- SV Bruksanvisning
- EL Οδηγίες Χρήσης
- HU Használati útmutatás
- PO Instrukcja stosowania
- HE הוראות שימוש
- JA 取扱説明書
- KO 사용법
- ZH 使用说明
- AR إرشادات الاستعمال



PARKER LABORATORIES, INC.



Parker Laboratories, Inc.  
286 Eldridge Road  
Fairfield, NJ 07004 USA  
(973) 276-9500  
parkerlabs.com



Parker Laboratories BV  
Bedrijvenpark Twente 305  
7602 KL Almelo  
The Netherlands

EN

Signaspray® Electrode Solution and Skin Prep

## Instructions for Use:

### Description

SignaSpray Electrode Solution and Skin Prep is a non-sterile, electrically conductive, non-gritty spray electrolyte. It is bacteriostatic.

### Indications for Use

SignaSpray is recommended for electromedical procedures (except defibrillation) or for use with muscle stimulation equipment.

### Contraindications

- SignaSpray is not indicated for defibrillation procedures.

*Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the EU Member State in which the user and/or patient is established.*

### Precautions

- For external use only on intact, unbroken skin.
- Discard product if package is damaged or evidence of contamination is present.
- To avoid contamination of product, spray nozzle or refill dispenser pump should not come into contact with the patient, staff or instruments.

### Directions for Use

Spray SignaSpray on the skin and apply the electrodes. Refer to the Instructions For Use provided by the manufacturer of the electromedical device being used for information on how SignaSpray should be used with their product. SignaSpray is water soluble and can be removed using a moistened soft cloth.

### Directions for Refilling Dispenser Bottles

Consult your facility policy before refilling from bulk dispenser container. For refilling spray bottles, wipe the bottleneck and dispenser tip with isopropyl alcohol or a hospital grade disinfectant, such as Protex™ Disinfectant Wipes, prior to filling. Reusable containers should be washed in hot soapy water, rinsed and thoroughly dried before being refilled. Only refill product into corresponding labeled spray bottle, and mark it with lot number and expiry date. Cracked or damaged bottles should be discarded immediately. Reusable bottles should **not** be topped off. Contents should be consumed, or the remaining contents should be disposed of prior to refilling.

### Handling

No special handling required.

### Storage

Protect product from direct sunlight.

### Disposal

Follow all federal, state, and local regulations for non-hazardous waste disposal.